

From: Barnard, Kathy
Sent: Friday, May 18, 2018 8:18 PM
To: Pearson, David A <David.Pearson@atriumhealth.org>; Colucciello, Stephen <Stephen.Colucciello@atriumhealth.org>
Subject: Stylet Cleaning process change

Hi,

After doing some infection prevention rounds we found that the stylets cannot be placed in these small purple bags and be placed at Major desk. We have ordered the large bio hazard bags seen in the picture below. This is how I would like to the process to go but need your approval and also need to share it with the providers. I will store the bags and pre-klenz in the trauma bays or even on the glide scopes so that they are easily accessible. Please let me know your thoughts.

After using Stylet, place in large bio hazard bag.

1. Spray stylet with Pre-Klenz and seal bag
2. Take bag to dirty utility room. Do not put on major desk.
3. HCT will walk down to Sterile Processing for cleaning.

Infection Prevention Rounds

- Stylet must go in larger bio bag (on order)



- Spray with Pre-Klenz



Kathy Barnard, RN
Nurse Educator
CMC-Main Adult Emergency Department
Office 704-355-8884
Ascom 704-446-2939

